

In-Check Dial Inspiratory Flow Trainer	University of Michigan Hospitals and Health Centers Infection Prevention & Epidemiology
Cleaning and Disinfection	
<i>IP&E approval dates: _12/2014_____</i>	

Where used:

Ambulatory Clinics, Inpatient Respiratory Care, Michigan Visiting Nurses

Frequency of Cleaning:

Must be cleaned and disinfected after each patient.

Responsibility:

Staff

Products:

Alcohol wipe or alcohol prep pad

Disposable one-way valve mouthpiece for In-Check Dial

Procedure:

1. Staff will attach one-way disposable mouthpiece to In-Check Dial
2. Patient will hold device. Patient's mouth should not touch In-Check Dial device (only the mouthpiece).
3. After use, staff will discard the one-way disposable mouthpiece.
4. Inspect the In-Check Dial device for internal contamination. If none, proceed. If contaminated, device must be discarded.
5. Staff will wipe exterior of In-Check Dial with alcohol wipe or alcohol prep pad.
6. Allow device to air dry.
7. Store in a protected area.

Authors: Kristen VanderElzen, Infection Prevention & Epidemiology

Karla Stoermer-Grossman, Pediatric Multi-Specialty Clinic