

## **Philips V-60 non-invasive Bi-PAP repurposed as a ventilator**

Institution: Beaumont Trenton

Author: Tom Deschamps, MHA, RRT  
Manager Respiratory Care  
Office 734-642-2229

- To use the Philips V-60 non-invasive bi-pap in the invasive mode or for those who are intubated, we made a change that is already built into the V-60.
- Under the menu function, we select Mask/Port.
- Once in this function, we then select ET/Trach.
- Before attaching to the patient's ETT, we place a filter inline between the bi-pap circuit and the patient.
- We have selected the AVAP's mode, to ventilate these patients.